AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear patient,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any

questions, please contact your doctor or medical retailer.

Different compression classes can be manufactured **for flat-knitted compression pantyhose**. The compression classes for the legs are then shown on the label.

Intended purpose

VenoTrain are medical devices. They are compression stockings for long-term treatment of chronic venous diseases, thromboembolism, edema and other indications (obesity with functional venous insufficiency, inflammatory dermatosis of the leg, pregnancy-related nausea and dizziness, congestion complaints in pregnancy, condition following burns, scar treatment).

Indications

Chronic venous disorders

- Varicose veins / initial period following varicose vein therapy
- Improvement of venous symptoms
- Prevention and treatment of venous edema
- Venous malformations
- Functional venous insufficiency (obesity, jobs involving sitting or standing)
- Improvement in quality of life for chronic venous diseases
- Prevention and treatment of venous skin changes
- Eczema and pigmentation
- Lipodermatosclerosis and atrophie blanche
- Prevention of recurring venous leg ulcers (CEAP stage 5)

Thromboembolism

- Superficial vein thrombosis
- Deep vein thrombosis
- Condition following thrombosis
- Post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients

Edema

- Lymphedema
- Phlebolympedema
- Edema in pregnancy
- Post-traumatic edema
- Postoperative edema
- Postoperative reperfusion edema
- Idiopathic cyclical edema
- Lipedemas
- Lipo-lymphedema
- Congestion conditions due to immobility (arthrogenic congestive syndrome, partial or complete paralysis of the extremities)
- Occupational edema (standing or seated occupations)
- Drug-induced edema, if no adjustment is possible

Other indications

- Obesity with functional venous insufficiency
- Inflammatory dermatosis of the leg
- Nausea and dizziness in pregnancy
- Congestion complaints in pregnancy
- Condition following burns
- Scar treatment

Part of the body this product is used for

In accordance with the indications

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist¹.

There are no known application risks if you use the product solely according to the indication and provided that you have been measured by a trained specialist and you have been taught the correct method for putting on the compression stockings. In the case of stockings with a silicone top band, skin irritation can occur in patients who are intolerant to this material. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful

to your health. In these cases, any warranty or liability claims will be excluded.

- Avoid contact with ointments, lotions, or substances containing grease or acids.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.
- Check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the affected part of the body. If the ground is smooth, there is an acute risk of slipping and thus injury!
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Contraindications

- Advanced peripheral artery disease (if one of these parameters is met: ABPI <0.5, arterial pressure at the ankle <60 mmHg, toe pressure <30 mmHg, or tcpO2 <20 mmHg at the instep)
- Decompensated heart failure (NYHA III + IV)
- Septic phlebitis
- Phlegmasia coerulea dolens

In the following cases, the decision to treat should be made after weighing the benefits against the risks in order to choose the most appropriate means of compression:

- Dermatitis with significant weeping
- Intolerance of the compression material
- Severely impaired sensitivity of the extremity
- Advanced peripheral neuropathy (e.g. in the case of diabetes mellitus)
- Rheumatoid arthritis

If you have any questions about this information, please contact your physician or medical specialist.

The manufacturer will not accept liability – particularly product liability – in the event of failure to comply with the above contraindications.

Application instructions

- The initial fitting must only be performed by medically trained specialists.
- It is best to put on your stockings first thing in the morning, as soon as you get up, or after lymph drainage.
- Wear Bauerfeind gloves to put on and take off your VenoTrain compression stockings as this makes it easier to distribute the material evenly.
- Your legs must be dry.
- Do not grasp the knitted fabric with sharp fingernails. Make sure that you take off your jewelry before starting.
- Please avoid overstretching the stockings and do not pull the product up by the silicone top band.
- Avoid damaging the knitted fabric and do not cut off any loose threads or labels, as this could cause stitches made during production to unravel.
- Store your VenoTrain compression stockings in a dry place away from direct light.
- Compression stockings are articles for daily use, and as such they are subject to natural wear when used on a regular basis. The recommended length of use is six months. This is based on correct handling (putting on and removal, care, etc.). The product's maximum useful life under proper storage conditions is indicated next to an "hourglass" symbol printed on the label on the packet.

Cleaning instructions

- Please use mild detergent and a lingerie bag. Never expose your product to direct heat. **Also observe the**

Instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

- When handwashing the stockings, we recommend drying them between two towels (do not wring).
- Avoid the use of washing additives like fabric softener, optical brighteners or stain removers.
- Please wipe the silicone non-slip band on the inside once a day with a damp cloth.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the VenoTrain have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

The following materials are used in our compression stockings: polyamide and elastane. In addition, the following compression stockings contain other materials: VenoTrain business (cotton), VenoTrain cocoon (cotton, cellulose) and VenoTrain hero (Merino wool). For information regarding the composition in percent, please refer to the individual sewn-in label. Some products can be fitted with a silicone top band. The top bands with silicone dots, silicone dot pattern, micro dots and the striped silicone top band are composed of polyamide, silicone and elastane, while the lace and lace sensitive top bands are composed of silicone, polyester, polyamide and elastane.

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Version: 2025-10

For more details, please consult your physician or medical retailer.

Tips for putting on the compression stockings

Closed toe

- 1) Turn the stocking "inside-out" up to the heel. Pull the foot part over your toes and then over the instep.
- 2) Fold the leg part back over and pull it over the heel at the same time. Make sure that the heel is in the correct position.
- 3) Now gradually slide the stocking upward and distribute it evenly until there are no creases. Never pull from the top or the top band.

Open toe

- 1) Use the blue VenoTrain donning aid supplied.
- 2) Gather the leg part together and pull the foot part over the donning aid. Make sure that the heel is in the correct position.
- 3) Now gradually slide the stocking upward and distribute it evenly until there are no creases. Never pull from the top or the top band. You can now remove the donning aid by withdrawing it from the open toe.

Please refer to the instructions provided on the first inside page of this brochure.

The use of the VenoTrain glider donning aid makes it easier to put on your VenoTrain compression stockings.

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of compression stockings which are relevant to you.

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