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ErgoPad® Soft

Polyurethan (PUR), Polyethylenterephthalat (PET)

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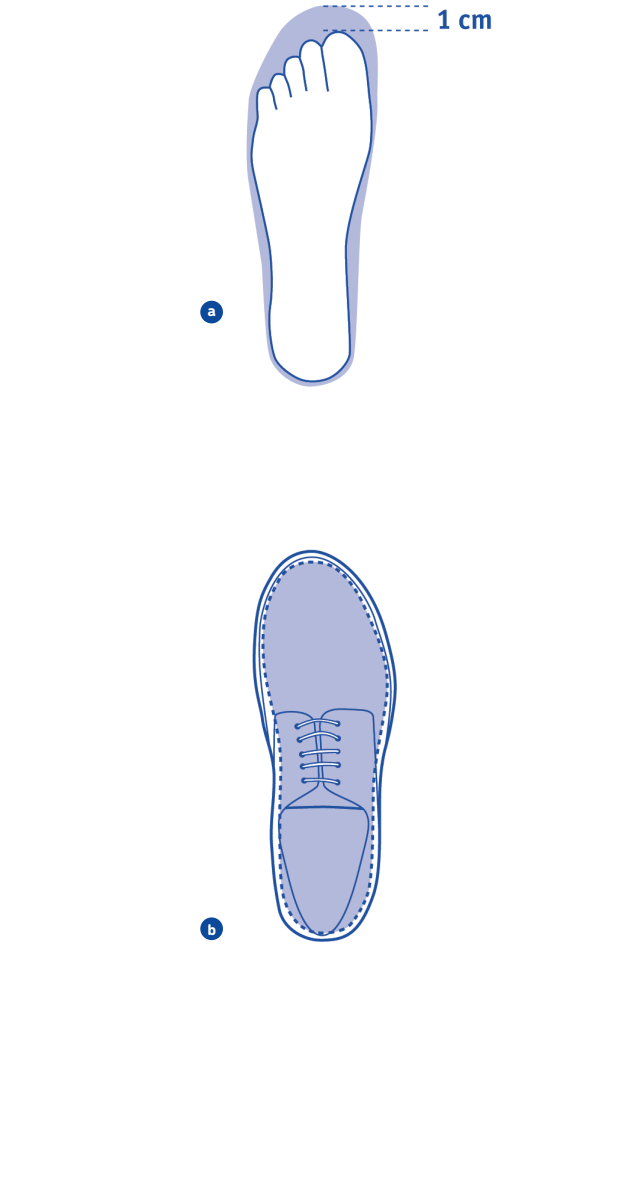
		
		
Contraindications		
Diabetic foot syndrome.		
Application instructions		
Successful application of the orthotic blanks depends on an appropriate type of foot orthosis being selected for the indication and intended use in question, the product being processed correctly and adapted to the wearer's shoes, and the wearer being given comprehensive instructions by specialists ¹ on how to use the product.		
Shoes suitable for wearing with foot orthoses must have sufficient internal volume for both the foot and the selected foot orthosis. An uncomfortable fit that constricts the foot must always be avoided.		
When selecting the correct size of foot orthosis, the packaging information serves only as a guide.		
Just like correctly fitting shoes, foot orthoses must be at least 1.0 cm longer than the foot, when the foot is in a weight-bearing, standing position. As feet may differ in length, please check both feet and, if in doubt, base your size selection on the longer foot. ⓐ		
The foot orthosis must be positioned straight in the shoe and have as much surface contact as possible with the shoe insole. Contact between the foot orthosis and the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet. ⓑ		
Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by your physician.		
The ErgoPad Soft foot orthoses are designed to be worn repeatedly by one user only.		
Please regularly check your foot for pressure marks, redness or other changes (caused by the foot orthosis). If you have any questions, please contact a specialist or your physician.		
Adjustment phase: As with many medical products, the ErgoPad Soft may feel somewhat different to wear at first. Normally, it should not take longer than a week to adjust to wearing the foot orthosis. If you still find it uncomfortable to wear after this period, please contact a specialist ¹ .		
Foot orthoses must be checked by a specialist after they have been worn for approximately six months to ensure that they are still functioning properly.		
Cleaning instructions		
The ErgoPad Soft can be cleaned by hand using soap and lukewarm water.		
Then leave the foot orthoses to air dry. Do not use any external heat sources (e.g. heaters, hairdryers, or direct sunlight). The foot orthoses cannot be washed in the washing machine.		
Part of the body this product is used for		
The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.		
Warranty		
Foot orthoses are items for daily use and as such they are subject to natural wear when used on a regular basis.		
Slight discoloration, in particular fading of the top covers of the foot orthoses, may occur during their use. This is dependent on a number of factors, e.g. moisture levels and sock / stocking materials. This is not covered by warranty.		
Warranty is provided in accordance with the provisions as well as the terms of supply and contractual terms of Bauerfeind AG (Terms and Conditions), as long as all the points mentioned in these instructions for use are observed.		
Disclaimer		
Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.		
Duty to report		
Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.		
Disposal		
Once you have finished using the product, please dispose of it according to local specifications.		
Material content		
ErgoPad® Soft		
Polyurethane (PUR), Polyethylene terephthalate (PET)		
 – Medical device  – DataMatrix Unique Device Identifier		
Version: 2023-04		
Specialists¹		
To ensure that the orthotic blank is used successfully and with an effective medical outcome, the correct type of foot orthosis must be selected by a specialist, who must also process it correctly and adapt it to the wearer's shoes. The wearer of the foot orthosis must also be given instructions on how to use the product. Orthotic blanks must be adapted to the wearer's individual foot conditions and shoes by grinding them or cutting them down and / or by thermoplastic molding. Thermoplastic molding: partial and complete thermoplastic molding is possible (in the oven at around 90 °C for 5 minutes)		
Grinding:	easy to grind (foot orthosis shape, edges of the foot orthosis) Caution: If the reinforcement element is damaged, breakages may occur or the function may be affected.	
Gluings:	with standard workshop adhesives recommended for foot orthosis material, used in accordance with the adhesive manufacturer's processing instructions	
Other:	a top cover, e.g. microfiber or leather, is recommended for mechanical protection and for hygiene reasons	

Contraindications

Diabetic foot syndrome.

Application instructions

Successful application of the orthotic blanks depends on an appropriate type of foot orthosis being selected for the indication and intended use in question, the product being processed correctly and adapted to the wearer's shoes, and the wearer being given comprehensive instructions by specialists¹ on how to use the product.

Shoes suitable for wearing with foot orthoses must have sufficient internal volume for both the foot and the selected foot orthosis. An uncomfortable fit that constricts the foot must always be avoided.

When selecting the correct size of foot orthosis, the packaging information serves only as a guide.

Just like correctly fitting shoes, foot orthoses must be at least 1.0 cm longer than the foot, when the foot is in a weight-bearing, standing position. As feet may differ in length, please check both feet and, if in doubt, base your size selection on the longer foot. **ⓐ**

The foot orthosis must be positioned straight in the shoe and have as much surface contact as possible with the shoe insole. Contact between the foot orthosis and the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet. **ⓑ**

Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by your physician.

The ErgoPad Soft foot orthoses are designed to be worn repeatedly by one user only. Please regularly check your foot for pressure marks, redness or other changes (caused by the foot orthosis). If you have any questions, please contact a specialist or your physician.

Adjustment phase: As with many medical products, the ErgoPad Soft may feel somewhat different to wear at first. Normally, it should not take longer than a week to adjust to wearing the foot orthosis. If you still find it uncomfortable to wear after this period, please contact a specialist¹.

Foot orthoses must be checked by a specialist after they have been worn for approximately six months to ensure that they are still functioning properly.

Cleaning instructions

The ErgoPad Soft can be cleaned by hand using soap and lukewarm water.

Then leave the foot orthoses to air dry. Do not use any external heat sources (e.g. heaters, hairdryers, or direct sunlight). The foot orthoses cannot be washed in the washing machine.

Part of the body this product is used for

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Warranty

Foot orthoses are items for daily use and as such they are subject to natural wear when used on a regular basis.

Slight discoloration, in particular fading of the top covers of the foot orthoses, may occur during their use. This is dependent on a number of factors, e.g. moisture levels and sock / stocking materials. This is not covered by warranty.

Warranty is provided in accordance with the provisions as well as the terms of supply and contractual terms of Bauerfeind AG (Terms and Conditions), as long as all the points mentioned in these instructions for use are observed.

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

ErgoPad® Soft

Polyurethane (PUR), Polyethylene terephthalate (PET)

 – Medical device
 – DataMatrix Unique Device Identifier

Version: 2023-04

Specialists¹

To ensure that the orthotic blank is used successfully and with an effective medical outcome, the correct type of foot orthosis must be selected by a specialist, who must also process it correctly and adapt it to the wearer's shoes. The wearer of the foot orthosis must also be given instructions on how to use the product. Orthotic blanks must be adapted to the wearer's individual foot conditions and shoes by grinding them or cutting them down and / or by thermoplastic molding.

Thermoplastic molding:	partial and complete thermoplastic molding is possible (in the oven at around 90 °C for 5 minutes)
Grinding:	easy to grind (foot orthosis shape, edges of the foot orthosis) Caution: If the reinforcement element is damaged, breakages may occur or the function may be affected.
Gluings:	with standard workshop adhesives recommended for foot orthosis material, used in accordance with the adhesive manufacturer's processing instructions
Other:	a top cover, e.g. microfiber or leather, is recommended for mechanical protection and for hygiene reasons

¹A specialist is any person who is authorized according to the state regulations that are relevant to you for fitting and instruction in the use of orthopedic orthoses.

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