BAUERFEIND AG
 Triebeser Straße 16
 07937 Zeulenroda-Triebes
 Germany
 P +49 (0) 36628 66-35 00
 F +49 (0) 36628 66-39 99
 E info@bauerfeind.com
BAUERFEIND.COM

ViscoHeel®

Gebrauchsanweisung
 Instruction for use
 Notice d'utilisation
 Istruzioni per l'uso
 Instrucciones de uso

**MADE
 IN
 GERMANY**

(de)	deutsch.....	2	(sr)	srpski.....	33
(en)	english.....	4	(sl)	slovenski.....	35
(fr)	français.....	6	(ro)	romanian.....	37
(nl)	nederlands.....	8	(el)	ελληνικά.....	39
(it)	italiano.....	9	(tr)	türkçe.....	41
(es)	español.....	11	(ru)	русский.....	43
(pt)	português.....	13	(et)	eesti.....	45
(sv)	svenska.....	15	(lv)	latviešu.....	46
(no)	norsk.....	17	(lt)	lietuvių.....	48
(fi)	suomi.....	19	(uk)	українська... ..	50
(da)	dansk.....	21	(kk)	қазақ.....	52
(pl)	polski.....	22	(he)	עברית.....	54
(cs)	česky.....	24	(ja)	日本語.....	56
(sk)	slovensky.....	26	(ko)	한국어.....	57
(hu)	magyar.....	28	(ar)	عربي.....	59
(hr)	hrvatski.....	30	(zh)	中文.....	60
(mk)	македонски ..	31			

(de) deutsch

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 english

Dear patient,

Thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

ViscoHeel is a medical device for the feet. It is a viscoelastic heel cushion. It provides shock absorption and relief during load peaks. ViscoHeel can also be used for the redistribution of pressure and for pain

reduction when treating pathologies in the heel and hindfoot areas.

Indications

- Achillodynia
- Tendomyopathy
- Haglund's deformity
- Tarsal bone pain
- Osteoarthritis of the knee
- Osteoarthritis of the hip
- Sacroiliac osteoarthritis
- Caudal intervertebral osteoarthritis
- Cement-free endoprosthesis care (in the first 12 months post-op)

Risks of using this product

Caution*

ViscoHeel must only be worn in accordance with the specifications in these instructions for use. To ensure the optimum fit of the ViscoHeel, the size required must be determined exactly. Only a professional¹ may carry out the (initial) fitting of the product and provide instruction.

Combination with other products, e.g. for evening out leg lengths, should be discussed beforehand with the physician treating you. If you notice any unusual changes or your symptoms get worse, please contact your physician without delay. No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids. Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed.

Contraindications

No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:

Skin disorders / injuries to the relevant part of the body, particularly if inflammation, impaired sensation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised

scars with swelling, redness, and excessive heat build-up

People with diabetes or rheumatoid foot deformities must seek medical advice before using this product to ensure optimum treatment

Application instructions

Inserting the ViscoHeel

Use the ViscoHeel in shoes with a stiff heel counter. Discuss the use of the ViscoHeel with the physician treating you beforehand, particularly if you want to use it in cushioned running shoes.

Unless otherwise prescribed by your physician, place **both** ViscoHeel heel cushions in your shoes so they fit snugly into the heel area of the shoes. Please take the right and left versions into account when doing this.

Cleaning instructions

Please note: Never expose the ViscoHeel to direct heat (e.g. heaters, sunlight, in your car, etc.!) This may cause damage to the material, which can impair the effectiveness of the ViscoHeel.

If necessary, you can clean your ViscoHeel heel cushions with a mild detergent in lukewarm water. Leave the heel cushions to air dry.

Notes on reuse

This product is intended to be fitted to one patient only.

Part of the body this product is used for

The heel cushions must be worn in suitable footwear and with socks or stockings on your feet.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be

limited or excluded if the instructions on how to handle and care for ViscoHeel have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional¹. Before first using our medical device, please seek the advice of a physician or professional¹ as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2025-06

* Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

¹ A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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Bauerfeind Ibérica, S.A.
Bauerfeind AG
Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.