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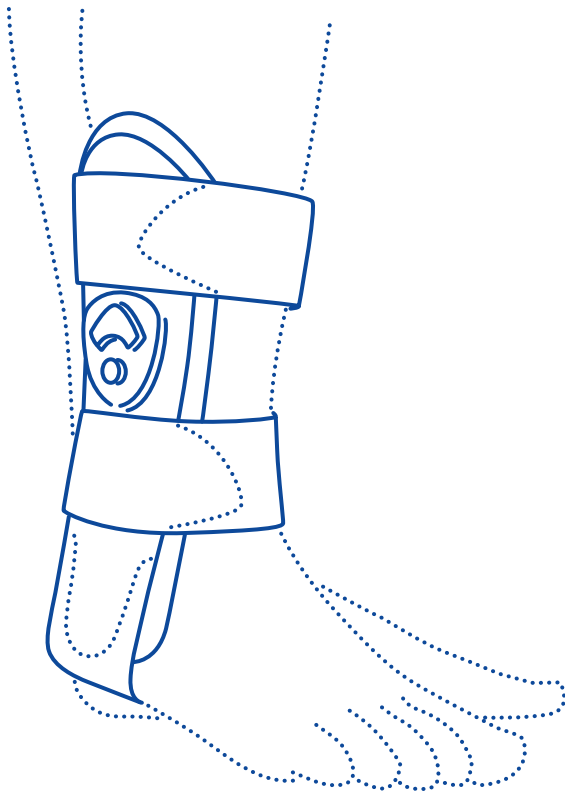
Dear patient,
thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and closely observe the specifications in these instructions for use and the instructions given by the specialist. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

AirLoc is a medical device. It is an orthosis¹ for the stabilization of the ankle with movement limitation in pronation and supination, including unilaterally adjustable air cushioning.

AirLoc must be individually adapted to the patient by a professional². This is the only way to guarantee the full effect and ideal wearing comfort of the orthosis.

Indications

- Treatment of (acute) injuries to the capsular ligament complex
- Early functional / non-surgical treatment, e.g. after ankle sprains and ruptured ligaments
- Post-operative rehabilitation / protection, e.g. after suture repair / ligament reconstruction
- Chronic ligament insufficiency / instability
- Recurrence prevention
- Weber A fracture (not dislocated)

Risks of using this product

Important Information³

- AirLoc is a prescribed product that should be applied under medical supervision. AirLoc should be applied in accordance with these directions for use and only for the specified indications.
- No liability is accepted for improper use of the product.
- The product **must be worn** in combination with **firm footwear** (e.g. sports shoes or outdoor shoes).
- No improper modifications may be made to the product. In case of non compliance, the performance of the product can be compromised. In addition, any product change will have a negative impact on questions of liability and could potentially lead to loss of liability.
- Patients with sensitive skin, known contact allergies or who are sensitive to pressure should wear a cotton sock or stocking under the AirLoc.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical provider.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Contact sports present a higher risk of injury. Bauerfeind does not guarantee that AirLoc will prevent injuries during these activities.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves. For this reason, check the fit of the product if you feel unpleasant pressure and consult the specialist in this regard or for general questions about handling. Should you notice increased symptoms or any unusual changes, please stop using the product and please contact your physician without delay.
Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical

professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Note: Never expose the AirLoc to direct heat (e.g. heater, sunlight, in your car, etc.)! This could damage the material. This can impair the effectiveness of the AirLoc.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Severe varicose veins.
- Impaired sensation and circulatory disorders of the legs, e.g. Diabetes mellitus.
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Application instructions

Putting on and taking off the AirLoc

Putting on the AirLoc

- Undo both straps and loosely refasten the ends of the straps.
- Open the cover of the valve on the outside of the orthosis and press the button below in order to release the air from the air cushion .
- Ensure there is no air left in the air cushions.
- Position the orthosis in such a way that the shell with the air cushion and the pump are on the outside of the affected leg .
- The shells are positioned on the side of the leg in such a way that the front edges of the orthosis are parallel to the edge of the shin .
- To fasten the orthosis, it is equipped with two straps on the shell.
- When putting the orthosis on the right leg, first the top strap is closed around the front and then the bottom strap is closed around the back .
- When putting the orthosis on the left leg, first the top strap is closed around the back and then the bottom strap is closed around the front .
- The Velcro ends are fastened to the middle of the relevant strap's fleece. The correct strap alignment is indicated by the contours on the shell at the fastening for the strap .
- When the orthosis is positioned correctly, the upper strap and then the lower strap must be re-tightened.
- Now put your shoe on.
- Pump up the air bladders using the integrated pump mechanism on the outside of the shell .
- The correct level of stabilization is usually achieved after 5–10 pump strokes. Please make sure you don't fill the air cushion with more than 20 pump strokes, otherwise it may be destroyed.
- The pressure can be individually adjusted via the pump and the release valve  + .
- To let out air, open the cover of the valve and press the button below into the shell in order to release the air .

Taking off the AirLoc

- Before taking off the shoe, open the valve cover on the orthosis. Empty the air bladders by pressing the button below .
- Now take off the shoe.
- To take off the orthosis, both straps must be opened.
- First open the top strap, then the bottom strap, and take off the orthosis.
- Before you put on the AirLoc again, please make sure the air bladder is empty.

Cleaning instructions

To prevent damage to the product, the AirLoc should only be washed by hand and maintained with a gentle cleaning agent / soap. Before washing, please fasten both Velcro straps to ensure their adhesive property is not affected.

Part of the body this product is used for

In accordance with the indications (ankle joint).

Assembly and fitting

AirLoc is supplied ready for use and does not need to be adjusted. It can be used on the left or right leg.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Technical parameters

AirLoc is an ankle orthosis with an inflatable air bladder and two adjustable Velcro straps.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the AirLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily.

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional². Before first using our medical device, please seek the advice of a physician or professional² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Once you have finished using the product, please dispose of it according to local specifications.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polypropylene (PP), Polyurethane (PUR), Polyamide (PA), Thermoplastic elastomer (TPE), Thermoplastic polyurethane (TPU), Polyethylene terephthalate (PET), Copper (Cu), Polyester (PES), Tin (Sn)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

² A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

³ Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

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Bauerfeind Benelux B.V.