AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

Dear patient,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please closely observe the specifications in these instructions for use and the instructions given by the specialist². If you have any questions, please contact your doctor or medical retailer.

Intended purpose

CaligaLoc is a medical device. It is an orthosis¹ for stabilization of the upper and lower ankle after serious injuries.

Indications

- Non-surgical treatment of torn ankle ligaments or a badly sprained ankle joint (distortion)
- Protection following reconstructive ligament surgery
- Permanent stabilization for complex instability of the upper and / or lower ankle

⚠ Risks of using this product

- CaligaLoc must only be worn in accordance with the specifications contained in these instructions for use and the areas of application (places of use) listed. To ensure the optimum fit of the CaligaLoc, it is essential to check whether the product is adequately shaped. If necessary, the CaligaLoc can be anatomically re-shaped by a specialist².
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product.
- The use and duration of use must be discussed with the physician who prescribed the CaligaLoc. They will also inform you whether the CaligaLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the CaligaLoc as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- **CAUTION!** CaligaLoc can only be worn in firm footwear. Otherwise, there is an acute risk of slipping and injury, particularly on smooth surfaces! If the product is used in different footwear or without shoes, the specialist must fully attach an anti-slip adhesive point to the

- pronation nubs that are knurled on the sole side.
 - Please check the floor and other walking surfaces to make sure they are non-slip before taking your first steps. Only run with extreme caution using short steps, remaining conscious of your ankle injury. Avoid slipping, as this could cause serious secondary injuries.
 - Furthermore, proceed with extreme caution, as your ability to stand and walk (for hand and arm products your ability to grip) is also heavily impacted.
 - Avoid contact with ointments, lotions, or substances containing grease or acids.
 - Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
 - If you experience any unusual changes connected to the use of the CaligaLoc – e.g. an increase in discomfort – stop using it and contact your physician without delay.
 - **Caution:** When loosening the straps or removing the product, there is not sufficient support for the injury. In this case, avoid any stress on the affected leg or foot.
 - Please adjust your behavior to the situation associated with your specific condition and any associated restriction of movement, and avoid anything that could delay recovery where possible. Proceed with particular caution.
 - Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- ⚠** Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Doff the orthosis if you notice any allergic reaction.

Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician. Ask your physician or supervising specialist which specific restrictions are personally relevant for you and must be observed. If your perceptive faculties are restricted, if you take painkillers or blood-thinning medication or suffer from dementia, our products must only be worn when prescribed by a physician and under constant supervision of trained personnel:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation and movement disorders of the legs / feet, e.g. in the case of diabetes mellitus
- Impaired lymph drainage, including soft tissue swellings of uncertain origin located remotely from the fitted orthosis

Application instructions

Putting on / fitting the CaligaLoc

A – plastic shell, B – upper strap, C – lower strap, D – middle Y-shaped strap

We recommend that hosiery should be worn under the CaligaLoc.

Sit down on a chair to put on the CaligaLoc. Place your foot on the floor with your foot at a 90° angle to your lower leg, as this is the best position for fitting the orthosis.

1 Undo all the straps. Place the relevant foot on the sole of the CaligaLoc, taking care to ensure the correct position of the heel. The outside of

the back of the foot should now be encased by the plastic shell. **2** Now wrap the upper strap round your lower leg from behind and guide it through the loop in the shin bone area. **3** Secure the end of the strap by fastening the Velcro to the section of strap which is underneath it. **4** Guide the lower strap over the instep of the foot to the outside of the foot and secure it in place with the Velcro. If necessary, the strap can be cut with scissors to make it shorter. **5** For the Y-shaped strap, proceed as follows: Guide the front strap in front of the shin bone and **6** the rear strap round the heel into the guide loops, then use the Velcro to secure them to the section of strap underneath. Tighten the straps again if necessary.

CaligaLoc can also be worn as a night splint while resting at night or during the day. Please remember to put on firm shoes as soon as you stand up and intend to put your full weight on the injured ankle while standing or walking according to your physician's advice.

Taking off / removing the CaligaLoc

Undo all the Velcro straps and lift your foot up and forward out of the orthosis.

Cleaning instructions

Note: Never expose the CaligaLoc to direct heat (e.g. heater, sunlight, in your car, etc.)! This can impair the effectiveness of the CaligaLoc.

Please refer to the cleaning instructions on the sewn-in label, which is located underneath the upper strap of the orthosis.

Wash the product by hand. Use a damp cloth to wipe the shell and cushion. Use a mild cleanser.

To make a complaint, please contact the retailer that provided you with the product.

Note: Only clean goods are handled.

Permissible conditions and places of use

In accordance with the indications (ankle joint).

Maintenance instructions

With correct handling and care, the product is practically maintenance-free.

Assembly instructions

The CaligaLoc is supplied ready for use.

Technical data / parameters

The CaligaLoc is an ankle orthosis with three attached straps. The straps are not removable.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the CaligaLoc have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical

specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polypropylene (PP), Polyamide (PA), Ethylene Vinyl Acetate (EVA), Steel, galvanized, Brass (CuSnZn), synthetic latex, Polyester (PES), Acrylate

Adhesive point

Elastodiene (ED), Acrylate (A)

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

Version: 2025-10

Specialists²

General notes

- Check the optimal and individual fit of the CaligaLoc at the first fitting. Make any corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect the medical and therapeutic effectiveness of the product, thereby voiding any product liability.

Adapting the CaligaLoc

Adjusting the plastic shell

- ❶ Check the fit of the CaligaLoc on the patient. Pay careful attention to the patient's individual anatomy (e.g. the lateral malleolus).
- ❷ Carefully remove the cushion from the plastic shell.
- ❸ If necessary, adapt the CaligaLoc using a hot-air gun at 120 – 140 °C (250 – 280 °F).
- ⚠ Wear protective gloves when you are doing this.
- ❹ Glue the inner cushion back into the plastic shell.

Adjusting the cushion

- ❶ If the ankle is badly swollen and the cushion is therefore pressing against the lateral malleolus, we recommend making a cutout in this part of the cushion. Use sharp scissors or a cutter knife and cut along the embossing.

Adjustments for use without shoes or without firm footwear

In case of a request to use CaligaLoc without firm footwear or without shoes, the enclosed anti-slip adhesive strip must be attached to the sole of CaligaLoc. The specific shape (make sure to trim!) and placement of the anti-slip adhesive strip on the underside of CaligaLoc must be adjusted

individually by the technician according to the patient's usage behavior. See Figure **A**.

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso
² A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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