CONTACTS

AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Dear patient,
Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

GenuTrain OA is a medical device. It is an orthosis¹ providing targeted relief and stabilization for more movement with osteoarthritis of the knee.

Indications

- Medial or lateral osteoarthritis of the knee
- Instability of the knee joint (mediolateral)
- Unicompartmental osteochondral lesions
- Condition after cartilage surgery and/or cartilage reconstruction for necessary unilateral post-operative relief
- Following unilateral meniscal lesions
- Osteochondritis dissecans (post-operative)
- Bone bruising (bone marrow edema of the medial femoral condyle / tibial plateau)

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the professional.

The benefits of this product are felt particularly during physical activity.

- Please doff your orthosis during longer periods of rest.
- Once you have been prescribed the GenuTrain OA, only use it according to your indication and any additional instructions given by a professional². Please consult a professional or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your physician.
- Caution: When loosening the straps or tensioning and fastening systems, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Furthermore, proceed with extreme caution, as your ability to stand and walk is also heavily impacted.
- Check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the affected part of the body. If the ground is smooth, there is an acute risk of slipping and thus injury!

Contraindications

No clinically significant adverse reactions have been reported to date.

If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Varicose veins (varicosis)
- Impaired sensation in the leg due to neurological abnormalities
- Circulatory disorders of the leg (e.g. in cases of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

- Before using the orthosis for the first time, practice donning and doffing it with a professional².

Donning of the GenuTrain OA

- To release the support straps, pull out the Boa® screw fastening ①.
- Take hold of the donning aids at the top on both sides, slip into the orthosis with an extended foot and pull it up and over the knee ②.
- Position the orthosis so that the kneecap is enclosed by the pad on the one side ③.

Setting the relief

- Press down on the screw fastening until it engages ④.
- Twist the screw fastening until you feel the desired relief ⑤. After a few steps, please be aware of the following:
 - The setting is too loose if the orthosis slips down or you cannot feel any relief or stabilization. Adjust the screw fastening if necessary.
 - The setting is too tight if you feel the support straps digging in. In this case, pull the Boa® Fit System out again to release the system, press down until it engages, and twist it to the desired degree.
- Adjust the relief according to your individual activities. TIP: If the orthosis “seems” to loosen after a few steps, the fastening can be readjusted later. While sitting or lying down, opening the fastening provides more comfort.

Only a professional may carry out the adjustment of the hinge

You can bend the hinge by hand without tools in order to better respond to the leg shape, reduce or increase the relief pressure ⑥.

Doffing of the GenuTrain OA

To release the support straps, pull out the screw fastening ①. Take hold of the removal aids at the bottom on both sides and pull the GenuTrain OA down ⑧.

Please note: If the movement of your GenuTrain OA was restricted by a professional, undo the front part of both circular straps before donning or doffing the orthosis each time ⑦. TIP: After undoing the straps, fold the Velcro tabs down again to avoid damaging the knitted fabric.

Cleaning instructions

- Please only wash the orthosis by hand at 30 °C using a mild detergent. All components can stay on the orthosis during washing. Leave the orthosis to air dry.
- Regular care guarantees the ideal effectiveness of the product. Also observe the instructions on the label underneath the removal aid.
- Never expose the product to direct heat as this may impair its effectiveness. If you have any complaints, please contact your distributor. Only clean goods are handled.

General information for professionals²

First fitting

- For medial relief, the flexible hinge can be found on the outside of the knee ①.
- For lateral relief, the flexible hinge can be found on the inside of the knee ②.

Adjust the splint anatomically, particularly for lateral relief. For stronger relief, you can adjust the splint to overcorrection. The hinge made from flexible spring steel can be adjusted to the desired shape by hand ③.

Part of the body this product is used for

Knee joint

Assembly and fitting instructions

The product is to be adjusted by professionals.

Maintenance instructions

If handled and cared for correctly, the product is maintenance-free.

Technical specifications / parameters

Your product comprises:

- Knitted mesh
- Boa® Fit System
- A lateral spring steel splint with a hinge
- Control elements made of plastic
- An incorporated pad around the kneecap and various cushions

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain OA have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Notice of liability

For use of the GenuTrain OA, the following applies:

Do not diagnose yourself or decide on which medication to take unless you are a professional². Before first using our product, please seek the

Setting the restriction (can be ordered as an option)

- Open the handle of the hinge cover and turn it a quarter turn until the indicator points toward the unlock symbol ⑩. Remove the hinge cover.
- Restrict the orthosis to 0°, 30°, or 60° extension and 30°, 60°, or 90° flexion ⑪.
- Use the limiting pins according to the desired limiting angle and use the same setting at the top and bottom. The limiting pin grips are above the hinge bolt ⑫.
- Use the straps to increase stability and secure the limitation of movement. Both straps each have two Velcro tabs between two guide loops ⑬. Attach the first Velcro tab of both straps to the velour area directly below and above the hinge ⑭. Turn the GenuTrain OA round to the other side and fasten the second Velcro tab to the velour there ⑮.
- Thread the straps through the guide loops on the first Velcro area, secure both straps and tighten them if necessary ⑯.

advice of this professional², as this is the only way to assess the effect of our GenuTrain OA on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of the professional² as well as all instructions in this brochure or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), stainless steel, Thermoplastic elastomer styrene-ethylene-butylene-styrene (TPE-SEBS), Silicone (Si), Polyurethane (PUR), galvanized spring steel wire, Elastane (EL), Ethylene-vinyl acetate (EVA), Polyketone (PK), Polyoxymethylene (POM), Cotton (CO), Thermoplastic polyurethane (TPU), Polycarbonate (PC), Polyester (PES), Iglidur polymer, Polytetrafluoroethylene (PTFE)

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

¹Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

²A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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