AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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english

Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Purpose

LordoLoc is a medical device. It is an orthosis¹ for the stabilization and relief of the lumbar spine using tensioning straps.

Indications

- General pain in the lumbar spine (acute and chronic)
- Moderate degeneration / muscular insufficiency of the spine

Risks of using this product

Important information

- LordoLoc is a product that should be applied under a physician’s guidance. LordoLoc should only be worn in accordance with these instructions for use and for the listed areas of application.
- The product should be worn over a T-shirt or an undershirt.
- LordoLoc must be individually adapted to the patient by a trained specialist². This is the only way to guarantee the full effect and optimal wearing comfort of the orthosis.
- Incorrect use of the support will terminate any product liability.
- No modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby terminating any product liability.
- Please consult your medical provider before using LordoLoc in combination with other products.

- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- If you have any complaints, please contact your medical retailer only. We would point out that only clean goods can be dealt with.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.
- For this reason, check the fit of the product if you feel unpleasant pressure and consult the specialist in this regard or for general questions about handling.
- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you notice any negative changes or an increase in symptoms while using the product, stop any further use and contact your physician immediately.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. If you are experiencing any of the following conditions, supports and orthoses such as the LordoLoc should only be fitted and worn after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation.
- Severe restriction of the performance of heart and lungs (risk of elevated blood pressure if the aid is worn during severe physical exertion).

Application instructions

Information for specialists²

If necessary, the stays can be adjusted.

Fitting LordoLoc

- Wrap the orthosis around your body so that the stays are positioned in the middle of your back. The tensioning straps are at the top **1**.
- Grasp the hand straps provided on the right and left of the fastening and pull the orthosis forward evenly **2**.
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the limbs or the torso **3**.
- When securing the Velcro fastening, remove the fingers from the left loop first, followed by the right loop **4**.
- Pull the two ends of the upper strap forward to produce the individually desired level of force to reinforce the compression **5**. Secure the ends of the upper strap against the base Velcro fastening **6**. Ensure that the whole Velcro section is fastened over the product so as to avoid contact between the Velcro tapes and clothing or skin.

Instructions for Product Care

Please use mild detergent and a mesh bag. We recommend closing the Velcro fastenings before washing to preserve their long-term adhesive strength and to prevent damage. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Part of the body this product is used for

In accordance with the indications (lumbar spine). See Intended use.

Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

LordoLoc is supplied ready for use.

Technical specifications / parameters

The LordoLoc is made from knitted fabric with additional tensioning straps. The back area of the orthosis is reinforced with corset stays. A fastening extension can be inserted to increase the circumference.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists². These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a specialist². Before first using our medical device, please seek the advice of a physician or specialist² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist² as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist², please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

High Grade Steel, Polyamide (PA), Polyurethane (PUR), Cotton (CO), Polyethylene (PE), Spandex (EL), Viscose (CV), Polyester (PES), Thermoplastic polyurethane (TPU)

– Medical device
 – DataMatrix Unique Device Identifier

Version: 2025-11

¹Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

²A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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