AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

Dear patient,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

LumboLoc is a medical device. It is an orthosis¹ with muscle-activating and relieving properties.

⚠ **LumboLoc must be individually adjusted for the patient by a trained specialist². This is the only way to guarantee the full effect and ideal wearing comfort of the orthosis. If the product is not used properly, any claim for warranty will expire.**

Indications

- General pain in the lumbar spine (acute and chronic)
- Moderate degeneration / muscular insufficiency
- Facet syndrome

⚠ Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

- The use and duration of use must be discussed with the physician who prescribed the LumboLoc. He will also inform you whether the LumboLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the LumboLoc as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- The product must be applied over a T-shirt or an undershirt.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Correct usage/fitting is assumed. Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves. For this reason, check the fit of the product if you feel unpleasant pressure and consult the specialist in this regard or for general questions about handling.
- If you experience unusual changes connected to the use of the LumboLoc – e.g. an increase in discomfort – stop using it and contact a physician immediately.
- Caution: When loosening the straps or tensioning and fastening systems, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we

recommend temporarily refraining from the activities mentioned above.

⚠ Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Side effects involving the body as a whole have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical advisor:

- 1.Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- 2.Impaired sensation.
- 3.Severe restriction of the performance of heart and lungs (risk of a rise in blood pressure if the aid is worn during severe physical exertion).

Application instructions

Fitting LumboLoc

- Wrap the product around your hips so that the stays are positioned on your back **1**.
- Place your fingers in the straps specially provided on the right and left hand fastenings and uniformly pull the support forwards **2**.
- Press the left hand fastenings against your abdomen and then pull the right hand fastenings until the two halves can be secured **3**.
- When securing the Velcro fastening withdraw your fingers from the left strap first, followed by the right strap **4**.

Cleaning instructions

We recommend washing the LumboLoc separately by hand at 30 °C using a mild detergent. To prevent damage and to preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing.

Note: Never expose the product to direct heat (e.g. heaters, sunlight, in your car, etc.))! This may cause damage to the material, which can impair the effectiveness of the orthosis.

Part of the body this product is used for

In accordance with the indications. See intended use.

Technical parameters

LumboLoc is made from knitted fabric with integrated dorsal corset stays.

Assembly and fitting instructions

LumboLoc is supplied ready for use.

Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. It is your first local point of contact before the manufacturer. Please use the LumboLoc in line with expert instructions and the enclosed instructions for use. Take note of the care instructions and do not make any alterations to the product. Only aids that

function perfectly will help you. Failure to follow the instructions may also compromise the warranty.

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Polyurethane (PUR), Spring Steel Strip, coated, Elastodiene (ED), Viscose (CV), Cotton (CO), Thermoplastic polyurethane (TPU), Polyester (PES), Spandex (EL)

MD – Medical Device

UDI – DataMatrix Unique Device Identifier UDI

Version: 2025-10

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.
² A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

Information for specialists²

If necessary the stays can be adjusted.

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