AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear patient,

Thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please closely observe the specifications in these instructions for use and the instructions given by the professional². If you have any questions, please contact your physician or medical retailer.

Intended purpose

LumboLoc Forte is a medical device. It is an orthosis¹ for increased stabilization and relief of the lumbar spine with a variable strap system.

Indications

- Moderate lumbosacral with slight muscular deficiencies in cases of intervertebral disk protrusions / herniated disks
- Moderate (pseudo) radicular lumbar syndrome
- Spondylolysis / spondylolisthesis, grade 1
- Moderate lumbar deformity

- Condition after herniation, e.g. post-operative care
- Ligamento-tendomyopathies
- Moderate degeneration / muscular insufficiency
- Osteoarthritis of the facet joints

⚠ Risks of using this product

- LumboLoc Forte is a product that should be applied under a physician's guidance. LumboLoc Forte should only be worn in accordance with these instructions for use and for the listed areas of application.
- The product can be worn over a T-shirt or an undershirt.
- LumboLoc Forte must be individually adapted to the patient by a professional². This is the only way to guarantee the full effect and optimal wearing comfort of the orthosis.
- No product liability is accepted in the event of improper use.
- LumboLoc Forte should not be altered arbitrarily.
- Please consult your physician before using it in combination with other products.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- Please first contact the retailer from whom you obtained the product in the event of complaints. Please note that only clean goods can be dealt with.
- Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.
- If you experience unusual changes connected to the use of the LumboLoc Forte – e.g. an increase in discomfort – loosen the straps and/or fastening a little or stop using it and contact a physician immediately.
- Caution: when loosening the straps or tensioning and fastening systems, or when doffing the product, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product.
- Restrictions may exist given the specific indication which the professional² can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

⚠ Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

 Contains natural rubber latex (elastodien), which can cause skin irritation in persons with an allergy. Doff the orthosis if you notice any allergic reaction.

Contraindications

No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up

- Impaired sensation
- Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is applied during intense physical exertion)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin in parts of the body where the orthosis is not worn

Application instructions

Donning and doffing the LumboLoc Forte

Donning:

- **1** Wrap the orthosis around your body so that it is positioned in the middle of your lower back (sewn-in labels at the top). The lumbar pad has to be mainly above the lumbosacral junction, and the sacral pad mainly below it.
- Place your fingers in the finger pockets **A** provided on the two broad, light-gray fastening halves **2** and pull the fastening forward evenly.
- **3** Lift your belly using the left fastening half and pull the right fastening half over it until you can attach them with the Velcro. When fastening the two halves, remove your fingers from the left-hand finger pocket first and then from the right.
- Grab the two LPT (low power tension) tensioning straps **4** with both hands **4** and pull them forward at the same time until the correct level of tension is created. Then attach the two LPT tensioning straps to the large fastening using the Velcro **5**.
- **Correct donning should be practiced with the patient.**

Doffing:

- Unfasten the LPT tensioning straps and loosely reattach them to the side of the product using the Velcro.
- Open the large, light-gray fastener.
- Doff the orthosis and refasten the light-gray fastener.
- Reattach the opened fastenings of the LumboLoc Forte to the product straight away for easier handling.

Cleaning instructions

- Please refer to the care instructions on the sewn-in label of your LumboLoc Forte. This can be found on the inner part of the knitted fabric.
- We recommend washing the LumboLoc Forte separately by hand at 30 °C using a mild detergent. To prevent damage and to preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing.
 - Separate the knitted fabric from the straps for cleaning.
 - Remove the pad for cleaning. The pad can be cleaned with a damp sponge.

Part of the body this product is used for

In accordance with the indications (lumbar spine).
Please see intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Technical specifications / parameters

A back orthosis made from knitted fabric with integrated dorsal corset stays, a height-adjustable strap and a back pad.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the LumboLoc Forte have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional² were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional². Before first using our medical device, please seek the advice of a physician or professional² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional² as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the professional², please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Polyurethane (PUR), Elastodiene (ED), Viscose (CV), spring band steel with PA coating, Polyoxymethylene (POM), spring steel wire, galvanized, Polyester (PES), Ethylene-vinyl acetate copolymer (EVAC), Polyethylene (PE)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2025-04

Information for professionals² / physicians

The functional strap **6** can be used in three different positions depending on the patient and indication.

- **1** The topmost position (in which the product is delivered) effectively reduces lordosis and lifts the torso.

- **II** The middle position leverages the torso to enhance lordosis and reduce a flat back.

- **III** To bring the product into the lowest position, pull the strap out of the guide on the left and right side of the product, turn it by 180° and put it back into the strap guide. This position reduces lordosis by tilting the pelvis back and straightening the torso. The hip is compressed.

Applying the pad

- Depending on the indication, the physician / professional can additionally apply one of either pad types (lumbar or sacral pad).
- The pads are attached to the fleece band provided on the inside of the orthosis using Velcro. When using the lumbar pad (Fig. A), the wide end of the pad must always face upwards. For the sacral pad (Fig. B), it is the other way around.
- When positioning the orthosis, take into account the anatomy (physique) of the patient. The pad can be reattached to a higher or lower position according to individual perception.

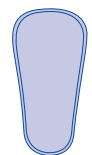
Fig. A
Lumbar pad

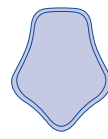
Fig. B
Sacral pad

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

² A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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