AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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(en) english

Dear patient,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

MOS-Genu is a medical device. It is an orthosis¹ for correction and stabilization after complex knee injuries or corrective osteotomy.

MOS-Genu must be individually adapted to the patient by a specialist². This is the only way the full effectiveness and ideal wearing comfort of the orthosis¹ are guaranteed.

Indications

- Non-surgical:
- Cruciate and lateral ligament injuries
 - Complex instabilities including genu recurvatum
 - Support for the affected compartment in bowleg / knock-knee arthritis

- Post-op. (short MOS-Genu):
- Operations on the ligaments, i.e. artificial ligaments
 - Meniscal suturing and meniscal replacement (meniscal implantation)

- Post-op. (long MOS-Genu):
- After axial correction of the shinbone (tibial osteotomy)
 - After complex reconstructions of the ligaments
 - In individual cases it is also indicated for fractures located close to joints

Risks of using this product

Important information

Please closely observe the specifications in these instructions for use and the instructions given by the specialist².

- MOS-Genu is a prescribed product that should be applied under a physician's guidance. MOS-Genu should only be applied in accordance with the instructions contained in these usage directions and only for the listed areas of application.
- No liability is accepted for improper use of the product.

- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- If you notice any changes in the product or an increase in symptoms while wearing the product, stop any further use and contact your physician immediately.
- A combination with other products (e. g. compression stockings) must be discussed beforehand with your medical provider.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- MOS-Genu is not designed to prevent injuries, especially when engaging contact sports. Contact sports inherently pose a greater risk of injury.
- If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves. For this reason, check the fit of the product if you feel unpleasant pressure and consult the specialist² in this regard or for general questions about handling.
- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product, including whether thrombosis prevention is necessary. Restrictions may exist given the specific indication which the specialist² retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Furthermore, proceed with extreme caution, as your ability to stand and walk is also heavily impacted.

 Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the orthose if you notice any allergic reaction.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Varicose veins.
3. Impaired sensation and circulatory disorders of the legs, e.g. diabetes mellitus.
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Application instructions

To make MOS-Genu as comfortable as possible for you to wear, please observe the following points:

- Keep the leg as clean as possible, particularly after the operation.
- Please put on and take off MOS-Genu whilst seated.

Putting on the orthosis

- Place MOS-Genu around your knee.
- Rotate round the knee pads until they are lying flat on the sides of the knee.
- Fasten the lower part of the orthosis as follows: Centre the main fixing strap incorporating the pad, pull on both sides simultaneously and secure the Velcro fastening. In the same manner, attach the strap below, beginning with the strap closest to the joint.
- Secure the upper part of the orthotic support as follows: First centre the strap closest to the joint, pull on both sides simultaneously and secure the Velcro fastening. Follow a similar procedure for the uppermost straps.
- Always pull the straps tight on both sides simultaneously.
- To ensure that the orthosis is fitted properly, use a mirror for verification.

Taking off the orthosis

⚠ IMPORTANT!

Unfasten the main fixing strap incorporating the pad last.

- First release all thigh straps, working from one side of the orthosis only.
- Then lift the upper part from the thigh. During this process the round pads at the sides remain in contact with the knee.
- Now, working from one side of the orthosis only, release straps on the lower leg section of the orthosis.
- After releasing all straps, slip the orthosis aforward and off the knee joint.

Product care

⚠ Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

The aluminium parts of the support are coated and can be cleaned with a pH-neutral soap. We recommend washing the cushions and straps by hand at 30 °C.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications (knee). See purpose.

Assembly and fitting instructions

The product is to be adjusted by specialists².

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Technical specifications / parameters

The MOS-Genu is a stabilizing orthosis for the knee joint. It consists of an upper and lower frame section, a hinge, cushions and straps.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MOS-Genu have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist² were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a specialist². Before first using our medical device, please seek the advice of a physician or specialist² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist² as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist², please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

For information regarding the options available for returning the packaging to ensure it is correctly recycled, please observe the instructions from the sales outlet where you purchased this product.

Material content

Aluminum (Al), Polyamide (PA), Polyurethane (PUR), Stainless steel (Fe), Ethylene-vinyl acetate (EVA), Cotton (CO), Polyethylene terephthalate (PET), die-cast zinc, chromed, Elastodiene (ED), Polyethylene, high density (PE-HD), Polypropylene (PP), Polyester (PES), Silicone (Si), Elastane (EL), Polytetrafluorethylene (PTFE), hot-melt adhesive (artificial rubber), polyether, acrylate

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2024-09

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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