AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

Dear patient,

Thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

ManuLoc Rhizo / ManuLoc Rhizo long are medical devices. They are orthoses³ for immobilization of the wrist (long version: and parts of the forearm) with a thumb brace.

If the product is not used properly, any claim for warranty will expire.

Indications

ManuLoc Rhizo

- Acute, post-traumatic and post-operative irritation in the wrist and thumb ray
- Pre-operative care
- Tenosynovitis at the radial side (moderate)
- Osteoarthritis of the first carpometacarpal joint (severe)
- Carpal tunnel syndrome (mild)
- Sprains/contusions
- Ligament injuries in the finger /hand area

ManuLoc Rhizo long

- Post-traumatic or post-operative irritation, e.g. following treatment with a plaster cast in the case of scaphoid fractures
- Pre-operative care
- Tenosynovitis at the radial side (moderate)
- Osteoarthritis of the first carpometacarpal joint (severe)
- Sprains/contusions
- Carpal tunnel syndrome (moderate)
- Collateral ligament injuries, e.g. skier's thumb

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the professional.

- ManuLoc Rhizo / ManuLoc Rhizo long must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application (intended purpose, parts of the body) listed.
- To ensure the optimum fit of the ManuLoc Rhizo / ManuLoc Rhizo long, the required size of ManuLoc Rhizo / ManuLoc Rhizo long must be determined by a professional².
- The sewn-in label of the ManuLoc Rhizo / ManuLoc Rhizo long, which contains information on the size, manufacturer, washing instructions and CE mark, is located in the insertion opening for the wide stay located on the inside of the hand (the volar stay). Only a professional² may carry out the (initial) fitting of the product and provide instruction.
- Use in combination with other products, e.g. as part of compression treatment, must be discussed beforehand with your physician.
- Should you experience increased discomfort or notice any unusual changes, stop any further use of the product please contact your physician without delay.
- No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed. Any aids – such as supports and orthoses³ – applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves. If this happens, loosen the straps of the orthosis a little and, if necessary, check the size of your orthosis.
- Caution: When loosening the straps or tensioning and fastening systems, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Contraindications

- No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:
- Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
 - Impaired sensation and circulatory disorders in the arm / hand, e.g. in cases of diabetes mellitus
 - Impaired lymphatic drainage – including unspecified soft tissue swelling located in distal direction of the fitted aid
 - Mental and physical disabilities that render it impossible to use the orthosis safely

Application instructions

Before using the orthosis for the first time, practice donning and doffing it with a professional².

Donning of the ManuLoc Rhizo / ManuLoc Rhizo long

➊ Start by loosening the three Velcro straps on the forearm splint (four Velcro straps for the long version). ➋ Place the ManuLoc Rhizo / ManuLoc Rhizo long on the correct hand. The straps should be located on the top of the forearm and on the back of the hand. ➌ Fasten the straps on the lower arm splint in the following sequence: Lower arm straps 1, 2 and the strap over the back of the hand (or lower arm straps 1, 2, 3 and the strap over the back of the hand on the long version). ➍ Close the Velcro closure on the thumb attachment.

Doffing of the ManuLoc Rhizo / ManuLoc Rhizo long

Open the lower arm straps, the strap over the back of the hand and the thumb closure. Doff the ManuLoc Rhizo / ManuLoc Rhizo long.

Cleaning instructions

Please note the following:

- Remove the Velcro straps if possible, or fasten them securely in position.
- Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.
- Please only wash the orthosis³ by hand at 30 °C using a mild detergent. All components can stay on the orthosis³ during washing. Leave the orthosis³ to air dry.
- Regular care guarantees the ideal effectiveness of the product. Also observe the instructions on the sewn-in label on the upper edge of your product.
- Never expose the product to direct heat as this may impair its effectiveness. If you have any complaints, please contact your distributor. Only clean goods are handled.

Part of the body this product is used for

In accordance with the indications (hand/ thumb). Please see intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The ManuLoc Rhizo / ManuLoc Rhizo long is supplied in an anatomically contoured state. If necessary, the ManuLoc Rhizo / ManuLoc Rhizo long can be shaped further. The orthosis is available in right- and left-handed designs.

Technical specifications / parameters

ManuLoc Rhizo / ManuLoc Rhizo long is an orthosis for the wrist and thumb. It consists of a long lower arm splint. The straps on the lower arm splint and the thumb support are detachable. The straps and thumb support are available as spare parts on request.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the ManuLoc Rhizo / ManuLoc Rhizo long have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional². Before first using our medical device, please seek the advice of a physician or professional² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the

responsible authority. Our contact details can be found on the back of this brochure. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Aluminum (AlMg3), Polyurethane (PUR), polyether, Polyoxymethylene (POM), Polyester (PES)

MD – Medical device

UDI – DataMatrix Unique Device Identifier UDI

Date of information: 2025-06

Professionals²

General instructions

- Check the optimal and individual fit of the ManuLoc Rhizo / ManuLoc Rhizo long at the first fitting, and adjust if necessary.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect the medical effectiveness of the product, thereby voiding any product liability.
- The ManuLoc Rhizo / ManuLoc Rhizo long is intended for the treatment of one patient only.

The aluminum stays are anatomically contoured, but they can still be individually adjusted to the patient.

Shaping the stays:

➊ Carefully remove the stays from the insertion pockets and shape them to the anatomy of the patient. Reinsert the three stays into the pockets provided.

Adjust the thumb attachment to the length and position of the patient's thumb:

➋ Loosen the lower Velcro closure between the hand orthosis and thumb attachment. Adjust to the patient's thumb length by moving the attachment within the channels provided. The thumb abduction angle is set directly by shaping the integrated aluminum stays and by the Velcro fastening on the thumb. Press the lower Velcro fastening down firmly after making adjustments.

¹ Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

² A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

³ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

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