AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Barrierefreie Version

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Aluminium (AlMg3), Polyamid (PA), Polyurethan (PUR), Polyether, Polyoxymethylen (POM), Polyester (PES), Polypropylen (PP)

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 english

Dear patient,

Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

The ManuLoc Rhizo long Plus is a medical device. It is an orthosis for immobilization of the hand with removable finger support and thumb brace.

Indications

- Post-traumatic care (multiple, complex injuries in the hand / finger area)
- Pre-operative and post-operative care, e.g. Dupuytren's contracture, metacarpal fractures
- Fractures / injuries in the carpal area of the wrist, e.g. following surgical treatment
- Injuries of the thumb saddle and first metacarpophalangeal joints
- Complex ligament injuries in the finger / hand area due to trauma
- Paralysis
- Tenosynovitis (severe)
- Sprains / contusions

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the professional.**
- Once you have been prescribed the ManuLoc Rhizo long Plus, only use it according to your indication and any additional instructions given by a professional². Please consult the professional or your physician first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected but may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
 - Avoid contact with ointments, lotions, or substances containing grease or acids.
 - Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
 - If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your physician.
 - Caution: when loosening the straps or tensioning and fastening systems, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
 - If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Contraindications

No clinically significant hypersensitivities have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation and circulatory disorders in the arm / hand, e.g. in cases of diabetes mellitus
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Donning of the ManuLoc Rhizo long Plus

- 1** Start by loosening the four Velcro straps on the lower arm splint, the Velcro strap on the finger support and the thumb section. **2** Place the ManuLoc Rhizo long Plus on the correct hand. The straps should be located on the top of the forearm and on the back of the hand. First, close the wide strap in the wrist area, then move up the arm, fastening the straps in order. Fasten the narrow strap on the back of the hand. **3** Then loosen one side of the finger strap, wrapping it around the finger joints and **4** velcro it to the underside of the finger support. **5** You can also fasten this strap in place to position the thumb securely.

Doffing of the ManuLoc Rhizo long Plus

Undo the straps on the lower arm splint, starting at the lower arm and finishing at the finger support. Take off the ManuLoc Rhizo long Plus.

Cleaning instructions

- Please note the following:
- Remove the Velcro straps if possible, or fasten them securely in position.
 - Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.
 - Please only wash the orthosis¹ by hand at 30 °C using a mild detergent. Leave the orthosis¹ to air dry.
 - Regular care guarantees ideal effectiveness of the product. Also observe the instructions on the sewn-in label on the upper edge of your product.
 - Never expose the product to direct heat as this may impair its effectiveness. If you have any complaints, please contact your distributor. Only clean goods are handled.

Part of the body this product is used for

see Indications and Intended purpose

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Technical specifications / parameters

- Your product comprises
- a metacarpal forearm splint (I)
 - removable finger support (II)
 - removable thumb brace (III)
 - removable straps on the forearm splint (IV).
- The thumb brace, the straps, and the finger support are available as spare parts.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the ManuLoc Rhizo long Plus have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Aluminum (AlMg3), Polyamide (PA), Polyurethane (PUR), polyether, Polyoxymethylene (POM), Polyester (PES), Polypropylene (PP)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2025-06

Professionals²

Assembly and fitting instructions

- Only a professional may carry out the fitting or assembly.
- Check the optimal and individual fit of the ManuLoc Rhizo long Plus at the first fitting, and adjust if necessary.
- Anatomically contoured aluminum stays can be individually adjusted to the patient. To do this, carefully remove the stays from the insertion pockets, shape them to the anatomy of the patient, and reinsert the three stays into the pockets provided.
- To adjust the thumb attachment to the length and position of the patient's thumb, loosen the Velcro fastening between the hand orthosis and thumb attachment (lower Velcro section). Adjust to the patient's thumb length by moving the attachment (within the channels provided). The thumb abduction angle is set directly by shaping the integrated aluminum stays and by the Velcro fastening on the thumb. Press the lower Velcro fastening down firmly again after making adjustments.
- The finger support is supplied in a neutral position and can be bent into the required position (e.g. the intrinsic plus position) depending on the treatment (at the height of the metacarpophalangeal joints).
- In the case of an assembled orthosis, adapt the dorsal extension angle in the wrist and the palmar flexion angle in the metacarpophalangeal joints. To do this, form

- the required support with your fingers in order to define the bending edge.
- Practice the correct procedure for donning of the product with the patient.
 - No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect the medical effectiveness of the product, thereby voiding any product liability.

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

² A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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