AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear patient,

Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

SecuTec Dorso is a medical device. It is an orthosis¹ for lordosis reduction and relief of the lumbar spine with the help of a bridging frame and corset stays.

 **SecuTec Dorso must be individually adapted to the patient by a professional². This is the only way to guarantee the full effect and ideal wearing comfort of the orthosis. If the product is not used properly, any claim for warranty will expire.**

Indications

- Spondylolisthesis (with relapsing lumbar sciatica) grade I–III
- Condition after prolapse (e.g. post-operative)
- Condition after discectomy
- Vertebral fractures (lumbar spine) with vertebral body damage
- Osteoporosis
- Moderate to severe lumbar sciatica with muscular deficiencies in cases of significant intervertebral disk protrusions/prolapse
- Severe (pseudo) radicular lumbar syndrome
- Moderate to severe lumbar deformity in cases of facet syndrome / osteoarthritis
- Lumbar spinal canal stenosis

Risks of using this product

 **Important information**

- SecuTec Dorso is a prescribed product that should be applied under a physician's guidance. SecuTec Dorso should only be worn in accordance with these instructions for use and for the listed areas of application.
- The product must be applied over a T-shirt or an undershirt.
- No improper modifications may be made to the product. Failure to comply may adversely affect the product's performance, thus canceling any product liability.
- Please consult your physician before using SofTec Dorso in combination with other products.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed. Any therapeutic medical aids – supports and orthoses – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, also constrict blood vessels or nerves. In such cases, please loosen the straps slightly and have the correct fit and size of the orthosis checked if necessary.
- If you use your product due to a medical prescription or acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Individual restrictions may exist which the professional² and also these instructions for use can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Adjust your behavior to the situation associated with your specific handicap / treatment need and any resulting restriction of movement, and avoid anything that could delay recovery where possible. Proceed with particular caution.
- If your perceptive faculties are restricted, if you take painkillers or blood-thinning medication or suffer from dementia, our products must only be worn when prescribed by a physician and under constant supervision of a professional².
- If you notice any negative changes or an increase in symptoms while using the product, stop any further use and contact a physician.
- Caution: when loosening the straps or tensioning and fastening systems, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part. Furthermore, proceed with extreme caution, as your ability to stand and walk (for hand and arm products your ability to grip) can also be heavily impacted.
- For products that affect the locomotor apparatus, check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the part of the body that is receiving treatment. If the ground is smooth, there is an acute risk of slipping and thus injury!

 Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Doff the orthosis if you notice any allergic reaction.

Contraindications

- If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:
- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Also in case of raised scars with swelling, redness and excessive heat build-up
 - Impaired sensation
 - Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is donned during intense physical exertion).

Application instructions

Information for professionals:

If necessary, the corset stays may have to be adapted.

Donning and doffing of the SecuTec Dorso

Donning

- Place the orthosis around your torso so that the stays interlinked in the grid system are positioned across the back, as shown in  (sewn-in labels at the top).

- Place your fingers in the finger pockets provided on the two broad, light-gray fastening halves and pull the fastening forward evenly.
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the Velcro fastening .
- When fastening the two halves, remove your fingers from the left-hand finger pocket first and then from the right .
- Take the two lower LPT (low power tension) tensioning straps in both hands and pull them forward at the same time to produce the desired level of force, then secure both LPT tensioning straps against the large Velcro fastening. Repeat this process for the upper LPT tensioning straps  until the orthosis is correctly fastened .
- Correct donning should be practiced with the patient.

Doffing

- Unfasten the LPT tensioning straps (first the upper followed by the lower straps) and then refasten them without any tension.
- Open the large, light-gray fastener.
- Doff the orthosis and refasten the light-gray fastener.
- Reattach the opened fastenings of the SecuTec Dorso to the product straight away for easier handling.

Cleaning instructions

- We recommend washing the SecuTec Dorso separately by hand at 30 °C using a mild detergent. To prevent damage and to preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).
 - Removal of the corset stays is not necessary.
 - If the corset stays are to be removed, bend the fasteners of the channels for the corset stays in the upper area and pull the stays out.
 - When reinserting the corset stays after washing, ensure that the corset stays are inserted into the correct pockets. You can easily tell which pocket is the correct one by the size. Make sure that the corset stays are completely covered by the channels and that the channels are closed again.
 - The stays must be inserted in such a way that the marking points upwards and outwards (away from the body).
 - Do not apply too much pressure on the stays so that their anatomical shape is not changed.
- This product has been tested by our own quality control system. However, should you have any complaints, please contact your local medical supply retailer.

Part of the body this product is used for

In accordance with the indications. Please see intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The SecuTec Dorso is supplied in a standard size and ready for use. The bridging frame must be individually adapted by a professional² and positioned in the knitted section.

Technical specifications / parameters, accessories

The SecuTec Dorso is an orthosis for the spine. It consists of a knitted section, a velour pocket for inserting the bridging frame and four tensioning straps with micro Velcro. The velour pocket and bridging frame are available as spare parts on request.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Where relevant, the country-specific warranty regulations between the distributor and buyer apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. Please do not make any alterations to the product yourself. This particularly applies to products that have been individually adjusted by a professional². Please follow our instructions for use and care.

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