AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear patient,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and closely observe the specifications in these instructions for use and the instructions given by the specialist. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

SofTec Dorso is a medical device. It is an orthosis¹ for increased stabilization of the thoracic and lumbar spine.

 **SofTec Dorso must be individually adapted to the patient by a specialist² in order to gain the maximum effect from this spinal orthosis.**

If the product is not used properly, any claim for warranty will expire.

Indications

- Non-surgical treatment (up to TH8) of simple and stable fractures
- Condition following discectomy
- Condition following spondylodesis
- Osteoporosis
- Tumors (metastases) or inflammatory processes (spondylitis)

Risks of using this product

 **Important information**

- SofTec Dorso is a prescribed product that should only be applied under a physician's guidance. SofTec Dorso should only be applied in accordance with the instructions contained in these usage directions and for the listed indications.
- The product should be applied over a T-shirt or an undershirt.
- The structure of SofTec Dorso may only be adjusted by a specialist². Failure to comply with this requirement may adversely affect product performance, thereby cancelling any product liability.
- Do not make any improper modifications to the product, including subsequent modifications to the knitted section. Failure to comply with this requirement may adversely affect product performance, thereby cancelling any product liability.
- Please consult your physician before using SofTec Dorso in combination with other products.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of

- a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- Side effects involving the body as a whole have not been reported to date. Correct usage / fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.
- In such cases, please loosen the straps or the fastening slightly and have the correct fit and size of the orthosis checked if necessary.
- If you notice any unusual changes or an increase in symptoms while wearing the product, stop any further use and contact your physician.
- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
-  Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the orthosis if you notice any allergic reaction.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. If you have any of the following conditions, such aids should only be fitted and applied after consultation with your physician:

- 1** Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, if you notice any raised scars with swelling, redness, and excessive heat build-up.
- 2** Impaired sensation.
- 3** Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is applied during intense physical exertion).

Application instructions

Fitting and removing the SofTec Dorso

Fitting:

- Before donning, the product is adapted to the patient by a specialist².
- Put on the orthosis like a backpack, passing your arms through the shoulder straps.
- It is important to ensure that the orthosis is correctly positioned 
- Place the fingers in the specially provided finger pockets on the two broad, light-gray fastener halves in the abdominal area and pull the fastener forward uniformly 
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the Velcro fastening 
- When securing the Velcro fastening, remove the fingers from the left pocket first, followed by the right.
- Take the two lower tensioning straps in both hands and pull forward at the same time to produce the correct level of force and secure both tensioning straps against the large Velcro fastening. Repeat this process for the upper tensioning straps 
- To achieve the desired correction / stability, place the thumbs in the specially provided blue loops on the pre-adjusted shoulder straps and pull the straps forward and down 
- When the upper body is straightened as required, secure the ends of the straps against the abdominal Velcro fastening 
- If you notice an unpleasant sensation of pressure below the shoulder strap, the accompanying pads can be fitted to the shoulder straps to absorb the pressure.
- Check the correct positioning of SofTec Dorso: The orthosis must be positioned as low as possible without pressing unpleasantly against the groin area while sitting upright.
- When first donning the orthosis, the optimal, individual fit must be checked, and corrected if necessary, by a specialist². Correct fitting should be practiced with the patient.

Removal:

- Unfasten the shoulder straps from the abdominal fastening and refasten further up on the shoulder strap.
- Unfasten the tensioning straps (first the upper followed by the lower straps) and then refasten without any tension.
- Open the large, light-gray fastener.
- Remove the orthosis and refasten the light-gray fastener.

- For easier handling of SofTec Dorso, always fasten any opened fasteners directly on the product.

Product care

⚠ Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

- We recommend washing SofTec Dorso separately by hand with a mild detergent at 30 °C. The aluminum frame of the back section must be removed prior to washing.
- The cross in the back section can be unscrewed with a 2.5 mm Allen key and the frame pulled upward out of the knitted section.
- To prevent damage and preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).
- The aluminum frame can be cleaned easily with a sponge and a pH-neutral soap.
- After washing, ensure that the frame, cross, and shoulder straps are correctly re-inserted.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications.

See Purpose.

Maintenance instructions

With correct handling and care, the product is practically maintenance-free.

Assembly and fitting instructions

The product is supplied as a pre-assembled orthosis in standard sizes. The stabilizing rods and connecting section must be positioned in the orthosis by a specialist².

Technical specifications / parameters

The SofTec Dorso is an orthosis for the spine. It consists of a knitted section, a velour pocket for inserting the rods, shoulder straps, and straps to adjust the straightening function individually. The reclinator, shoulder pads, cross, and inner and outer knitted section are available as spare parts on request.

Notes on reuse

The SofTec Dorso is intended to be fitted to one patient only and may not be passed on to any other user.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function perfectly will help you. Failure to follow the instructions may also restrict the warranty.

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a specialist². Before first using our medical device, please seek the advice of a physician or specialist², as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device

to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Aluminium (Al), Polyurethane (PUR), Polyoxymethylene (POM), High Grade Steel, Acryl Isocyanate, Elastodiene (ED), Polyester (PES), Polypropylene (PP), Cotton (CO), Polyethylene (PE), Spandex (EL), Polyether, Steel galvanized, Acrylate

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

Version: 2025-10

¹Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or torso

²A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

Specialists:

Adaptation of SofTec Dorso:

- Shape the one-piece reclinator to the necessary curvature (angle of lordosis / kyphosis) according to the indication and the patient's physique **❶**
- The material should only be bent in large arches (no sharp bends).
- The material can be molded repeatedly.
- If bending hooks are employed, only round bending hooks should be used.
- Insert the reclinator in the velour channels on the back section and guide the rods over the small blue pockets integrated in the knitted section so that the rods project from the product below **❷**.
- Fit the product to the patient so that the neck section ends at the shoulder and the lower edge of the knitted section reaches down to the groin. Check the fit of the pre-shaped frame against the patient **❸**.
- Mark the required length on the rods projecting downwards out of the velour pockets and trim to length **❹**.
- Now insert the shoulder straps (Polybag B) in the product. To do this, secure the ends of the straps with the blue thumb pockets to the light gray abdominal fastening **❺**. Pull each strap backwards and upwards through the lateral velour channel at the top edge **❻**. The straps must be crossed over the back. Pass the right strap forward again through the left-hand slot on the neck section. Do the same with the left strap on the opposite side **❼**. Now pull each of the front ends through the topmost reversing buckle, from the inside to the outside **❽**, and secure against the Velcro spot on the respective strap **❾**.
- The upper fastener of the shoulder strap is fastened roughly in the area between the reversing buckle and the collarbone and is thereby pre-adjusted for the patient (trim the strap if necessary).
- Now remove the reclinator from the product and the shoulder straps from the neck section to shape the neck section to the patient's anatomy.
- The additional aluminum cross (Polybag A) can be fitted to the frame using the accompanying assembly kit (four drive-in nuts and four cap screws M4 × 6).
- The angle of curvature (lordosis angle) of the cross must first be shaped anatomically to the frame **❿**. Next, the pre-drilled holes are marked at the required level on the rods (observe the gaps in the velour, use the cross as a template). Remove the reclinator from the product again to press the supplied nuts into the holes so that their collars are located on the side facing the body. Finally, screw the cross to the re-assembled product (reclinator, knitted fabric and shoulder straps).
- During the final assembly, the rods must be inserted into the small blue pockets integrated in the lower end of the rear velour channels. This stops them from slipping.

- Check the fit of the orthosis on the patient in both a standing and a sitting position after each modification.
- The patient should practice the correct way to put on the orthosis with help from the orthotist.

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