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Dear patient,
Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully.
If you have any questions, please contact your physician or medical retailer.

Intended purpose

Spinova Mum is a medical device. It is an orthosis¹ for the relief of the lumbar spine and stabilization of the sacroiliac joint during and after pregnancy (postpartum regeneration).

Indications

Lumbar spine indications:

- Pain and instability in the pelvic / lumbar spine area
- Low back pain
- Radicular / pseudo-radicular lumbar syndrome
- Acute or chronic lumbosciatica
- Moderate lumbar deformity (e.g. hyperlordosis)
- Prolapse

Other indications:

- Symphysis relaxation
- Symphysis rupture
- SI joint pain
- Sacroiliac joint disorders (SI joint syndrome, SI joint arthrosis, SI joint instability)
- SI joint blockage (acute and prophylactic)

Risks of using this product

⚠ Caution*

Please closely observe the specifications in these instructions for use and the information provided by professionals².

- If you experience premature contractions, you can still use the product, but **without** pads.
- The fundamental suitability and use of the orthosis during and after pregnancy is to be determined by a physician. If you experience any discomfort while wearing the orthosis during pregnancy, immediately doff the product and consult your physician without delay. They will decide on whether and how you should continue using the product.
- Size adjustment and donning technique should be practiced and reviewed regularly with a professional².
- Your Spinova spinal orthosis must be utilized in accordance with the specifications contained in these instructions for use and only for the areas of application (intended purpose, part of the body this product is designed for) listed.
- To ensure the best possible fit and ideal function of the orthosis, a professional² must provide instruction concerning the product. No product liability is accepted in the event of improper use.
- If you notice any changes or your existing symptoms get worse, stop using the product and please contact your physician without delay. Doff the orthosis for longer rest phases.
- In case the buckles cause irritation while sitting, loosen the orthosis to avoid pressure points and pull the orthosis tight as soon as you stand up again.
- Please consult a professional² or your physician first if you want to use it together with other products.
- Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves. Avoid fastening the strap too tightly below the belly. If you feel an intolerable pressure, please consult the physician who prescribed the product. If you still find the pressure uncomfortable, please consult your physician or professional².
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the physician or professional² can clarify. In particular, the ability to operate

vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

- If you are wearing the orthosis after surgery (e.g. Caesarean section), the wound must be checked regularly by a physician.

Use during pregnancy

The fundamental suitability and use of the orthosis¹ during and after pregnancy is to be determined by a physician. If you experience any discomfort while wearing the orthosis¹ during pregnancy, immediately doff the orthosis and consult your physician without delay. They will decide on whether and how you should continue using the product. Size adjustment and donning technique should be practiced and reviewed regularly with a professional².

Contraindications

Spinova Mum should not be used if you have an inguinal hernia or serious spinal diseases (e.g. fractures, tumors, spondylolysis) that require further orthopedic care/treatment.

If you have any of the following conditions, the orthosis should only be donned and worn after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation and movement disorders, e.g. in the case of diabetes mellitus

Application instructions

Please refer to the care instructions on the sewn-in label.

The sewn-in label contains information on the size, manufacturer, care instructions, and CE marking and is located on the inside of the orthosis.

Spinova Mum can be worn on the skin or over lightweight clothing.

Note: Never expose the product to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may damage the material, which can impair the effectiveness of the orthosis.

Donning

- 1 Guide your right hand into the hand strap of the right abdominal fastening **(A)**, making sure that the sewn-in label **(H)** is positioned on the upper edge. Place the orthosis around your back and guide your left hand into the hand strap on the left abdominal fastening.
- 2 Stretch the material of the Spinova Mum on both sides and position the orthosis centrally on your back.
- 3 Now place your left hand under your abdomen and gently support it upward.
- 4 Then guide your right hand under your abdomen and attach the right fastening by pulling on the left side and remove your left hand from the hand strap at the same time. In the early months of pregnancy, the abdominal fastening should be closed very tightly, but as time goes by, less tightly.
- 5 When the abdominal fastening is closed, hold the tensioning straps on the right and left **(B)**.
- 6 By pulling the tensioning straps to the side, the abdominal lifting **(C)** can be adjusted individually.
- 7 By pulling the tensioning straps toward the front, the hip compression of the non-elastic strap **(D)** can be adjusted individually.

Please note that the abdominal lifting must always be adjusted first, and then the hip compression. During this process, the pulling force on the tensioning strap must be maintained until it is fixed on the abdominal strap. If the pulling force varies in the meantime, the level of force may change.

8 + 9 As soon as the level of force has been adjusted, the tensioning straps are fastened onto the abdominal fastening under tension.

While wearing the Spinova Mum, the tensioning straps can be loosened at any time and the level of force can be changed. This

may be necessary when switching from a sitting activity to a standing activity, or vice versa.

Pads

The pads consist of 2 individual components.

For reinforced stabilization and massage of the back, the 2-part pads **(J)** are secured on the dorsal side of the Spinova Mum. These support the stabilizing effect of the orthosis and the relaxing effect in the area of the back and pelvis with massage nubs. The pads are fastened on the velour with Velcro patches **(I)**, see above) on the inside of the orthosis. The bottom edge of the pad normally ends where the bottom edge of the orthosis begins. On the back of the pads, there are arrows pointing upwards to facilitate proper orientation. The pads can be positioned individually by a professional².

The pads can be removed and attached depending on the sensation and condition.

Doffing

Detach the tensioning straps **(B)** from the abdominal fastening **(A)** and attach them to the strap on the side. Then reach into the hand straps of the abdominal fastenings **(A)**, pull them away from each other and take off the orthosis. Make sure to reattach the abdominal fastening **(A)** after doffing to avoid damage to the material.

Part of the body this product is used for

In accordance with the indications. See intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Components

- A** – Abdominal fastening with hand straps
- B** – Tensioning strap with Velcro
- C** – Elastic abdomen lifting strap
- D** – Non-elastic hip strap
- E** – Corset stays
- F** – Mesh material
- G** – Strap
- H** – Sewn-in label
- I** – Velour for positioning pads or compresses
- J** – Pads

Assembly and fitting instructions

Spinova Mum is delivered ready to use in the basic version.

The corset stays **(E)** at the back can be individually shaped as needed. The pads must be positioned individually.

Technical specifications / parameters

Spinova Mum consists of mesh material **(F)** with abdominal fastenings **(A)**, tensioning straps **(B)** and dorsal corset stays **(E)**. The tensioning strap is used to control an elastic abdominal lifting strap **(C)** and an non-elastic hip strap **(D)**. To ensure that the straps are positioned correctly, a strap guide **(G)** is attached. On the inside, velour strips **(I)** are attached in order to fix the pads **(J)** in place using Velcro.

Notes on reuse

This product is intended to treat one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer directly from whom you obtained the product in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the product have not been observed. Warranty is excluded if:

Cleaning instructions

Please refer to the care instructions on the sewn-in label.

Clean the orthosis at regular intervals. You can wash the Spinova Mum by hand at 30 °C with a mild detergent. Before doing so, close all the Velcro fastenings in order to prevent damage to the material and remove the pads with Velcro patches. Leave the orthosis to air dry.

The pads can be washed by hand at 30 °C with a mild detergent. The Velcro patches need to be removed before cleaning the orthosis and must not be washed with the orthosis. Leave the pads to air dry.

- The product was not used according to the indication
- The instructions given by the professional² were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional². Before first using our medical device, please seek the advice of a physician or professional² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional² as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics, etc.). If you still have doubts after your consultation with the professional², please contact your physician or distributor, or contact us directly.

Duty to report

Because of regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the

manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Polyurethane (PUR), Polyoxymethylene (POM), Polyester (PES), Elastane (EL), spring steel wire, Polyethylene (PE), Thermoplastic elastomer (TPE), Polypropylene (PP), Cotton (CO)

MD – Medical device

UDI – DataMatrix Unique Device Identifier UDI

* Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

² A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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